**anodynesurgical.**

PSI/EYE-KO, Inc. dba Anodyne Surgical

804 Corporate Centre Drive, O'Fallon, Missouri 63368 USA

800-428-5628 | 636-447-1010

[anodynesurgical.com](http://anodynesurgical.com)

Instructions for use

PVA EYE SPEAR

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HEMOSTETA OFTÁLMICA

# English

## **1. Product Description:**

The PVA EYE SPEAR is a sterile, single use device consists of a spear and a handle. The spear is made of PVA (polyvinyl alcohol) material which can absorb aqueous fluids several times of its mass instantaneously upon contact with the fluids. The handle is made of polypropylene plastic material.

## **2. Intended Use:**

The PVA EYE SPEAR is indicated for use in ophthalmic or microscopic surgical procedures for absorbing excess fluids from the operative field.

## **3. Caution:**

- 3.1. This device should be used on the order of a physician.
- 3.2. For single use only.
- 3.3. Do not use if the bag is broken or damaged.
- 3.4. Do not reuse, as it may result in compromised device performance and risk of cross contamination.
- 3.5. PVA EYE SPEAR may not be moistened with an alcoholic solution, as then the structure of the material will change.

## **4. Warnings:**

- 4.1. Sterility guaranteed unless unit package is opened or damaged.
- 4.2. Do not attempt to use the spear for cleaning of the surgical sites or removing debris, such as vitreous material that might be generated during the surgery.

## **5. Prohibit:**

The PVA EYE SPEAR is not for use as an applicator for applying medication to the surgical sites. Also, the PVA EYE SPEAR shall not be used as a hemostasis tool for compressing the spear against the bleeding site.

## **6. Contraindications**

- 6.1. Absolute Contraindication:  
Patient allergy to PVA (polyvinyl alcohol) materials.
- 6.2. Relative Contraindication:  
Pregnant woman or the patients who have had any adverse reactions by using PVA related materials.

## **7. Use Procedure:**

### **INSTRUCTIONS FOR OPENING THE PRODUCT PACKAGE**

- 7.1. The product package contains two foiled pouches sealed in an envelope. The envelope has a Tyvek top layer and a clear plastic bottom layer.
- 7.2. To open the package, separate the clear plastic layer from the Tyvek layer and gently pull open the envelope.
- 7.3. Empty the two foiled pouches onto a sterile field.  
Caution: Maintain sterile technique when contacting the inner foiled pouches.

### **INSTRUCTIONS FOR OPENING THE STERILE FOILED POUCHES**

- 7.4. Each foiled pouch contains five (5) PVA EYE SPEAR.
- 7.5. While maintaining sterile technique, tear open the foiled pouch.
- 7.6. Remove spears and place in the sterile field.  
Caution: In order to avoid inadvertent contact of the spear with aqueous fluids, keep the spears in a dry place within the sterile field.

### **INSTRUCTIONS FOR USING THE PVA EYE SPEARS**

- 7.7. By holding the plastic handle, gently touch the tip of the spear at the site where excess fluid is to be removed.
- 7.8. Hold still for approximately 3 seconds to allow for complete swollen of the PVA material.  
Caution: Do not squeeze or press the PVA eye spear against the surface of the surgical sites.
- 7.9. Dispose of the used spear into a biological waste container.
- 7.10. When necessary, use additional spears to absorb fluid until all excess fluid is satisfactorily removed.

## **8. Preservation & Handling:**

- 8.1. The product should be preserved in a dry, well-ventilated place and in sealed conditions.
- 8.2. Do not use if package is opened or damaged and the identification tag with the Lot no. should not be separated while in preservation.
- 8.3. Storage condition: 0°C~ 37°C.

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