anodynesurgical.

PSI/EYE-KO, Inc. dba Anodyne Surgical

804 Corporate Centre Drive, O'Fallon, Missouri 63368 USA

800-428-5628 | 636-447-1010

anodynesurgical.com

Instructions for use
Ophthalmic Sponge

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English

1. Product Description:

The Ophthalmic sponge is a sterile, single use device. The Ophthalmic sponge is made of PVA (polyvinyl alcohol) material which can absorb aqueous fluids several times of its mass instantaneously upon contact with the fluids.

2. Intended Use:

The Ophthalmic sponge is indicated for use in ophthalmic or microscopic surgical procedures for absorbing excess fluids from the operative field.

3. Caution:

- 3.1. This device should be used on the order of a physician.
- 3.2. For single use only.
- 3.3. Do not use if the bag is broken or damaged.
- 3.4. Do not reuse, as it may result in compromised device performance and risk of cross contamination.
- 3.5. The Ophthalmic sponge is not suitable to be moistened with an alcoholic solution as the structure of the sponge will be changed.
- 3.6. The Ophthalmic sponge is not suggested to use on people who may be allergic to PVA sponge.

4. Contraindications

- 4.1. Absolute Contraindication:
Patient allergy to PVA (polyvinyl alcohol) materials.
- 4.2. Relative Contraindication:
Pregnant woman or the patients who have had any adverse reactions by using PVA related materials.

5. Warnings:

- 5.1. Sterility guaranteed unless unit package is opened or damaged.
- 5.2. Do not attempt to use the sponge for cleaning of the surgical sites or removing debris, such as vitreous material that might be generated during the surgery.

6. Preservation & Handling:

- 6.1. The product should be preserved in a dry, well-ventilated place and in sealed conditions.
- 6.2. Do not use if package is opened or damaged and the identification tag with the Lot no. should not be separated while in preservation.

6.3. Storage condition: 0°C~ 37°C.

7. Prohibit:

The Ophthalmic sponge is not for use as an applicator for applying medication to the surgical sites. Also, the ophthalmic sponge shall not be used as a hemostasis tool for compressing the spear against the bleeding site.

8. Use Procedure:

8.1. Eye Drain:

- 8.1.1. Remove adhesive backing and with wick(s) securely to drape for gravity drainage.
- 8.1.2. Moisten wick(s) with sterile fluid and insert around eye site (without pinching).
- 8.1.3. Irrigate normally.

8.2. Eye Wick:

- 8.2.1. Moisten wick(s) with sterile fluid and insert around eye site (without pinching).
- 8.2.2. Irrigate normally.

8.3. Wipe / Wipe & Wick:

- 8.3.1. Moisten wipe with sterile fluid thoroughly.
- 8.3.2. Clean fine instruments with moisten wipe.
(Caution: Be careful of sharp instruments, avoiding cut or punches hands or fingers while cleaning.)

8.4. Cleaning Blocks:

- 8.4.1. Moisten one side of block with a few drops of sterile water. Thoroughly rinse knife with sterile water.
- 8.4.2. Pierce wet side of block with knife blade several times.
- 8.4.3. Pierce dry side of block several times.
(Caution: Excessive bending may damage blade.)

8.5. LASIK Eye Drain & LASIK Ring:

- 8.5.1. Remove LASIK Eye Drain/LASIK Ring and place in the sterile field. Moisten LASIK Eye Drain/LASIK Ring if needed.
- 8.5.2. Use the LASIK Eye Drain/LASIK Ring at the surgical process site where excess fluid is to be removed.

8.6. Corneal Light Shield:

- 8.6.1. Remove Corneal Light Shield and place in the sterile field. Moisten Corneal Light Shield if needed.
- 8.6.2. Place the Corneal Light Shield onto the corneal to prevent light illumination and maintain cornea moist throughout surgery.

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